



WiseUp KEYCHAIN BREATHALYZER

Features

- 4-stage breath-alcohol tester (warm-up, blow, analysis, result)
- Quick response (total time of stages - 20 seconds)
- Bright white LED flashlight function

Battery Installation

- Slide down battery compartment lid on the back of the unit.
- Insert two AAA batteries with correct polarity as indicated.
- When lights become dim, replace with new batteries.

Important information

User will press the power button continuously through all 4 stages of breath test. If finger slips at any point in the 20 second cycle, do not re-start for 3 minutes (while sensor resets itself). Failure to use as directed will cause inaccurate results.

Stages of breath alcohol test

- warm up – 10 seconds
- blow – 5 seconds
- analysis – 4 seconds
- result – 1 second

Breath alcohol test

Press and hold the Power button, the Green light will flash for 10 seconds (warming up) and then becomes solid Green (now entered blow stage). User must blow steady (and not too hard) into pipe for 5 seconds (begin blowing immediately upon solid green light - the 5 second blow period has started and will not wait for you). The final 2 seconds of exhaust from lungs is needed by sensor for optimal Breath Alcohol Content (BAC) reading. All three lights will now blink for 4 seconds while BAC analysis occurs. BAC range result is now revealed in one single solid light. Test now complete. Do not use the device again for minimum 3 minutes (while unit clears and re-sets itself for future test).

Flashlight function

- Set the switch to ON position to turn on flashlight.
- Do not use the flashlight function during BAC testing.

Precautions

- Make sure to press Power button continuously for 20 seconds through the stages.
- It is recommended that test subject not drink, eat or smoke for 20 minutes prior to test.
- Wait 3 minutes between tests, or results will not be accurate.
- Do not use chemicals to clean the device.
- Do not blow liquid into the breath-pipe.

Disclaimer

WiseUp does not warrant the accuracy of the readings of the breathalyzer. It is not intended that the customer rely on the device to provide the customer with an accurate blood alcohol reading; it is intended merely to provide guidance to the customer. The customer therefore uses the breathalyzer at his or her own risk. To use the breathalyzer effectively, the customer must strictly adhere to the instructions. Any deviation from such may have the effect of decreasing the accuracy of a reading.